





# Jorgensen Laboratories, Inc.

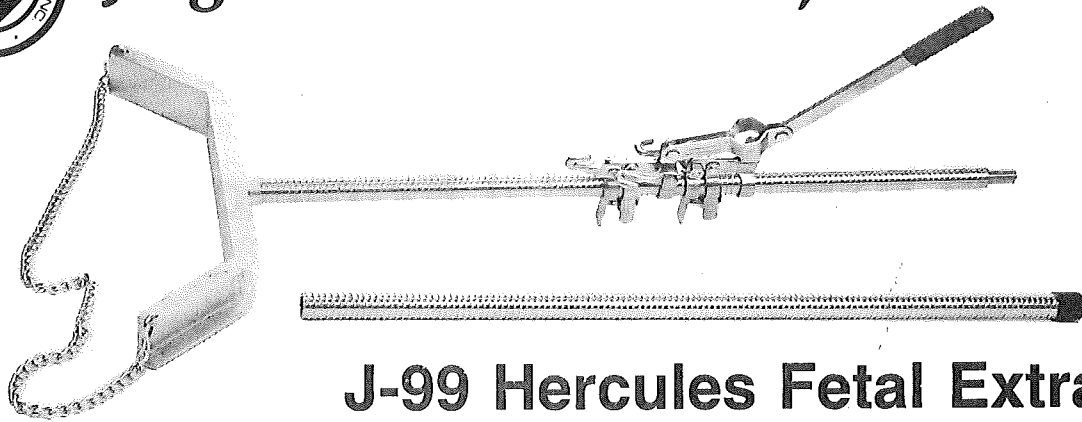

## J-99 Hercules Fetal Extractor

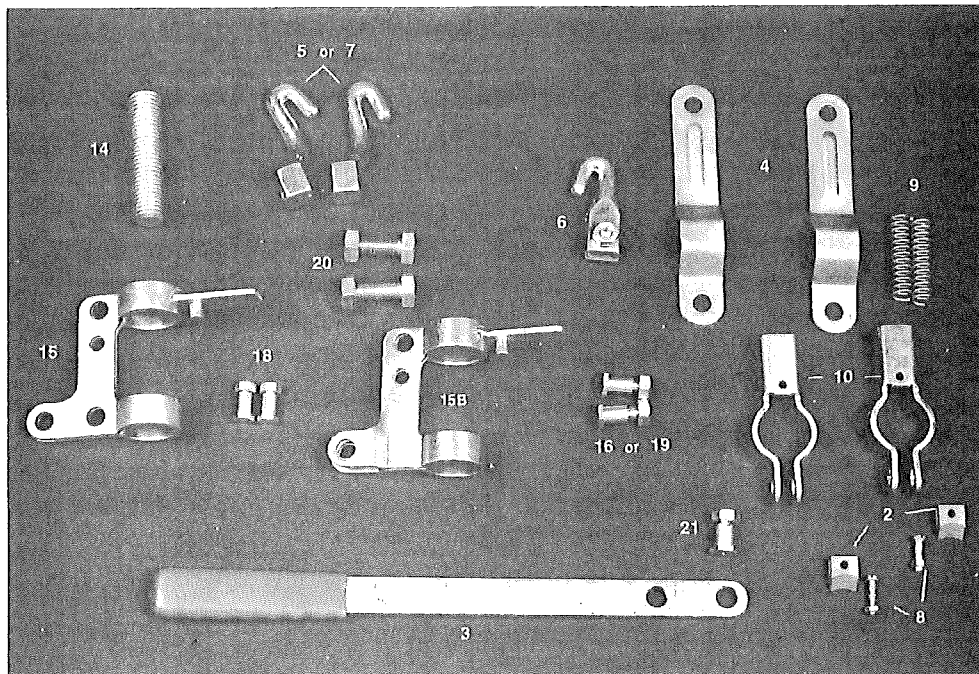

### J-99 FETAL EXTRACTOR

J-99d1	Ratchets (pair) - USA model (not pictured)	J-99d11B	½ shaft (rubber cap end, not pictured)
J-99d2	Wear plates (pair) - German model	J-99d12	Breech plus chain & S-hooks (not pictured)
J-99d3	Handle	J-99d13	Complete jack (not pictured)
J-99d4	Side bracket (pair)	J-99d14	Connecting stud for shaft
J-99d5	Hook (left or right side)	J-99d15	Front double-eye bracket. (holds 2 hks)
J-99d6	Middle hook	J-99d15B	Back double-eye bracket
J-99d7	Hook (right or left side)	J-99d16	Bolt & nut for side bracket
J-99d8	Wear plate screw & nut (each)	J-99d17	Breech chain with 2 S-hooks (not pictured)
J-99d9	Compression spring (each)	J-99d18	Bolt & nut for double-eye bracket
J-99d10	Wear plate support (each)	J-99d19	Bolt & nut for wear plate support
J-99d11	Complete shaft (not pictured)	J-99d20	Bolt & nut for side hooks
J-99d11A	½ shaft (breech portion, includes 14, not pictured)	J-99d21	Bolt & nut for handle or middle hook

The Hercules calf puller provides a new approach to difficult calvings. The ratchet type jack has 3 separate hook attachments, one for each of the two obstetrical leg chains, and a separate hook for the head sling. Two of these hooks pull alternately along the shaft by moving the jack handle back and forth. Even a very difficult birth can be helped by using this zig zag pull.

To release under pressure, walk jack backward by releasing one catch at a time.

The extra long shaft (65") can be disassembled in the middle, and is made from thick walled steel pipe. Deep grooves guarantee against slipping.

Breech is supplied with an easy to clean adjustable chain.

**DISTRIBUTED BY:**



# Jorgensen Laboratories, Inc.

## J-99 Hercules Fetal Extractor

Congratulations on your purchase of our J-99, Hercules calf puller. This is the original model as designed in West Germany. The original unit has been modified and strengthened to fit the demands of the U.S. market.

The Hercules calf puller offers the user the following features:

1. The shaft screws apart at the breech and mid-point for easy storage. Be sure the shaft is screwed all the way together at the mid-point or the shaft may bend.
2. The jack assembly has three hooks. By putting one chain on one side hook and another chain on the other side hook, you will be pulling one leg at a time. This alternating or "zig-zag" pull actually "walks" the calf out. This causes less strain on the pulled calf.
3. The middle hook can be used for a chain around the calf's head or if you want to pull both legs simultaneously.
4. The calf puller should be taken apart and cleaned thoroughly after each use.
5. To release under tension, it is necessary to release and move backwards one catch at a time.
6. The following accessories are available from Jorgensen Laboratories and will ease your effort in pulling calves:
  - a. J 24-WL or J 24-WS, 60-inch or 30-inch plated steel chains.
  - b. J109, J-Lube lubrication jelly.
  - c. J 99-S Cable type head snare.

Note: For a complete listing of Jorgensen products, see your local distributor.

### **WARNING**

Please consult your local veterinarian for proper use of this device. Use of this equipment without veterinary supervision may result in severe trauma, injury or death to the cow, mare, calf or foal. Veterinary supervision is recommended.

It is recommended that you familiarize yourself with the operation of this calf puller before using.