


### Auscultation Trainer and SmartScope™ LF01142U Instruction Manual

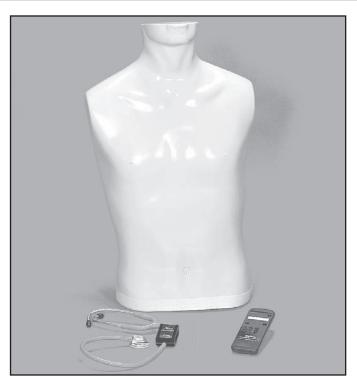

### *Life/form* Products by Nasco

#### About the Simulator

The *Life/form*<sup>®</sup> Auscultation Trainer Simulator duplicates heart and lung conditions selected by the instructor by wireless remote control. The student must palpate to identify the correct auscultation sites and will hear different heart and lung sounds as the SmartScope<sup>™</sup> is moved from site to site. The simulator duplicates human conditions as closely as modern plastics and electronic technology allows — it's almost the real thing. Care and treatment should be the same as with a patient.

The simulator has six heart sites and four lung sound sites on the anterior surface, with 14 lung sound locations on the posterior. The remote control can select from 12 different heart conditions, as well as 16 lung conditions. The instructor can select any condition, then switch to another condition easily so the student can compare sounds and make a diagnosis.

The remote control does not have to be pointed directly at the manikin or stethoscope to operate. One remote control will operate multiple sets of SmartScopes<sup>™</sup> and manikins simultaneously. Great for group instruction. The range of the remote control is up to 100 feet.

#### **General Instructions for Use**


To begin using the Auscultation Trainer, press the red power button. This turns on the remote control and sends a signal to activate the stethoscope as well. After the unit is activated, the display will be in the "status" mode, displaying the current menu settings for the heart and lung conditions. To select a new condition, press either the heart or lung button. This will put the display into the menu mode. The user can select a condition by either using the number buttons, or by viewing the conditions in sequence using the scroll button. After the condition is selected, press the enter button to activate.

# The remote control can be programmed to these heart conditions:

- 01 Normal
- 02 Aortic regurgitation
- 03 Pulmonary stenosis
- 04 Mitral stenosis
- 05 Holosystolic
- 06 Mid-systolic
- 07 S3 Gallop
- 08 S4 Gallop
- 09 Systolic click
- 10 Atrial septal defect
- 11 PDA
- 12 VSD

# The remote control can be programmed to these lung conditions:

- 01 Normal tracheal
- 02 Normal vesicular
- 03 Wheezes
- 04 Mono wheeze
- 05 Fine crackle
- 06 Coarse crackle
- 07 Ronchi crackle
- 08 Stridor
- 09 Cavernous
- 10 Bronchovesicular
- 11 Bronchial
- 12 Pulmonary edema
- 13 Infant
- 14 Friction rub
- 15 Egophony
- 16 Pectoriloquy

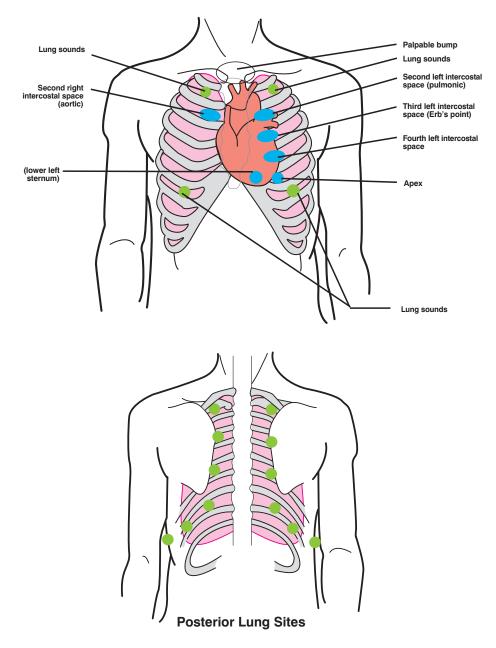

**Batteries:** The remote operates on 2 "AAA" batteries. Remove sliding cover on lower back to replace as needed.

SmartScope<sup>™</sup> operates on 2 "AAA" batteries. Remove sliding cover on lower back to replace as needed.

### Other Available *Life/form*. Simulators

LF00698U Adult Injectable Arm (White) LF00855U Male Catheterization **LF00856U** Female Catheterization LF00901U Prostate Examination LF00906U Ostomy Care LF00929U Surgical Bandaging LF00957U Enema Administration LF00958U Pediatric Injectable Arm **LF00961U** Intramuscular Injection LF00984U Breast Examination LF00995U Arterial Puncture Arm LF00999U Pediatric Injectable Head LF01005U First Aid Arm LF01008U Intradermal Injection Arm LF01012U Heart Catheterization (TPN) **LF01019U** Ear Examination LF01027U Peritoneal Dialysis LF01028U Suture Practice Arm LF01034U Suture Practice Leg LF01036U Spinal Injection LF01037U Hemodialysis Practice Arm LF01038U Episiotomy Suturing Set LF01042U Suture Kit LF01062U Pelvic, Normal & Abnormal LF01063U Stump Bandaging, Upper LF01064U Stump Bandaaina, Lower LF01069U Cervical Effacement LF01070U Birthing Station **LF01082U** Cricothyrotomy LF01083U Tracheostomy Care LF01084U Sigmoidoscopic Examination LF01087U Central Venous Cannulation LF01095U Blood Pressure Arm LF01108U Infant Intraosseous Infusion LF01121U Advanced IV Arm LF01131U Venipuncture and Injection Arm LF01139U Advanced IV Hand LF01142U Auscultation Trainer LF01143U Testicular Exam LF01152U Male & Female Catheter LF01155U Advanced CPR Doa LF01162U Venatech IV Trainer LF01174U NG Tube & Trach Skills

ulaiors	
LF01184U	Venatech IM & Sub Q
LF01193U	
LF03000U	. ,
LF03601U	
EIUUUU	Trainer with Stand
LF03602U	Adult Airway Management
1.000020	Manikin
LF03609U	Child Airway Management
	Trainer with Stand
LF03616U	
LF03617U	Deluxe Child <b>CRiSis</b> ™
	Manikin with Arrhythmia Tutor
LF03620U	PALS Update Kit
LF03623U	
	Trainer with Stand
LF03632U	Child Intraosseous Infusion/
	Femoral Access Leg on a Stand
LF03633U	
	Trainer Torso
LF03693U	Basic Buddy <sup>®</sup> CPR Manikin
LF03699U	"Airway Larry" Airway
	Management Trainer
LF03709U	Infant <b>CRiSis</b> ™ Manikin
LF03720U	Baby Buddy <sup>™</sup> Infant CPR Manikin
LF03750U	Fat Old Fred
LF03760U	Airway Management/Cricoid
	Pressure Trainer
LF03770U	Chest Tube
LF03953U	
LF03955U	Deluxe <b>CRiSis</b> ™ Manikin
LF03956U	Deluxe "Plus" <b>CRiSis™</b> Manikin
LF03965U	
	Manikin
LF03966U	Adult <b>CRiSis™</b> Auscultation
	Manikin with ECG Simulator
LF04000U	
LF04200U	
	Infusion
LF06001U	CPR Prompt <sup>®</sup> Adult/Child
	Manikin
LF06012U	CPR Prompt <sup>®</sup> Infant Manikin
	CPR Prompt <sup>®</sup> Keychain
	Rescue Aid
LF06204U	CPR Prompt <sup>®</sup> Rescue and
	Practice Aid

Ordsco Fort Atkinson
901 Janesville Avenue, P.O. Box 901
Fort Atkinson, Wisconsin 53538-0901
 1-800-558-9595
eNasco.com ● E-mail: lifeform@eNasco.com
COPYRIGHT © NASCO 2000 PRINTED IN U.S.A. NP 30-00/RV 8-12